
Simeon Talkback

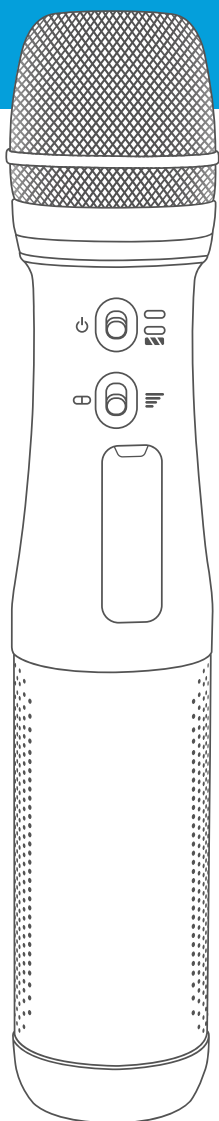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WELCOME TO TALKBACK

Talkback is an all-in-one mini handheld amplification system. It combines a microphone, amplifier and speakers, all in a single pass-around unit. Talkback is ideal for small group meetings, kindergarten circle time or any small venue where voice reinforcement is needed.



Using the device

Talkback is ready to use right out of the box. Included as standard* within the packaging are:

- Talkback handheld unit
- Carrying case
- USB charging cable with power adapter
- Drop-in external charging cradle
- Lanyard strap


*Other optional accessories are available to expand the possibilities of your Talkback device

mic+amp +speaker

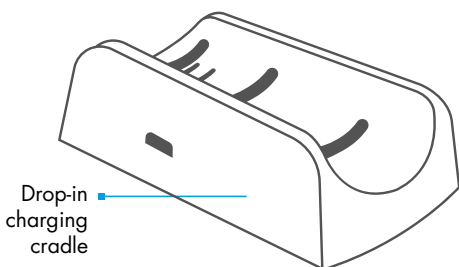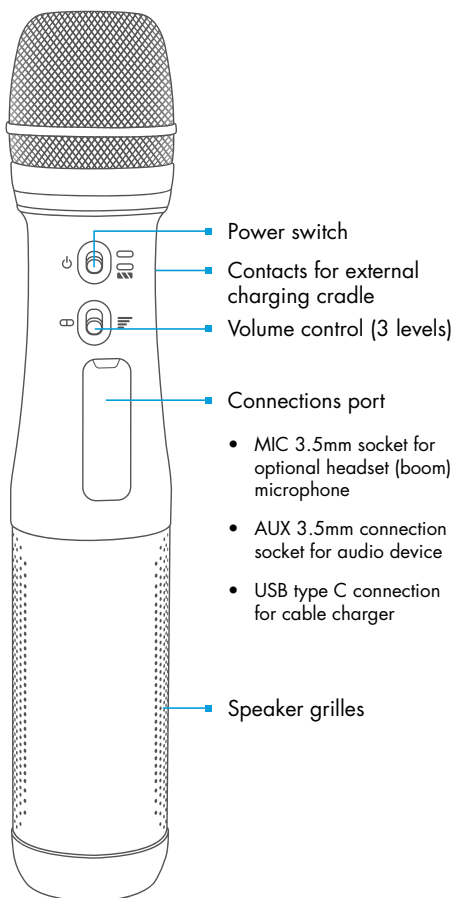
For best sound, position the microphone close to your mouth and keep it away from your body.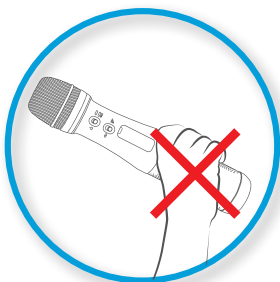

Do not cover the speaker grilles at the bottom of the unit with your hands.

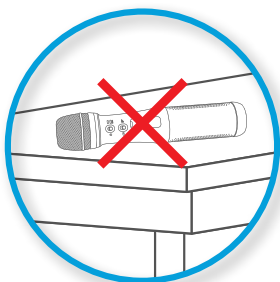To avoid feedback (squealing noise) do not set the unit on a table or hard surface when it is turned on.

mic+amp +speaker

Precautions

- Do not use the device next to the ears of other people.
- Select the volume level according to the size of venue. Small spaces require less volume.
- Feedback (squealing noise) may occur if the volume level is too loud for the room space or if the unit is placed too close to hard, reflective surfaces. Turn the unit off before placing it flat on a tabletop.
- Use only compatible accessories.
- Do not immerse the unit in water or expose it to heat.
- The unit is best sanitized in a UV-C sterilization cabinet that is suitable for electronic equipment. Alternatively, it may be surface cleansed with a cloth or wipe containing 70% isopropanol, taking care to avoid fluid seeping into the microphone or speaker grilles. Allow to dry thoroughly before use or before charging.

Warranty

Your Simeon Talkback device has a one year warranty as standard, effective from the date of purchase. The manufacturer's warranty provides for repair or replacement of defective components or workmanship. Expressly not covered are damages resulting from water damage and the unapproved substitution of components, batteries or accessories. Extended warranty coverage may be purchased under the Simeon Service Plus extended service program.

Specifications

Rated output	10W
Frequency response	200Hz - 15KHz ± 2 dB
Dimensions	26 cm overall length
Weight	300g
Charging	DC 5V (charging cradle); USB C-Type
Talk time	≤ 6 hours
Charge time	2.5 hours

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Supportive Hearing Systems Inc.